

587 Seat Height Combinations

Using only the seat height post setting, the 587 has a 4" seat height adjustment range in 1" increments.

Seat height settings which are outside of the 4" range permitted by the seat height post setting are achieved by changing the mag wheel axle mounting position and/or the mag wheel diameter. Subsequently, the caster wheel axle mounting position on the fork will need to be adjusted to maintain a level base orientation. If the caster wheel axle position on the fork cannot be adjusted enough to maintain a level base orientation, the caster wheel diameter will need to be changed.

Padding supplied with the strapped seat will add approximately 1/2" to the uncompressed strapped seat height.

All dimensions are approximate.

Seat Height ⁽¹⁾	Mag Wheel Diameter	Mag Wheel Axle Mounting Position	Caster Option (Axle Position)	Seat Height Post Setting 2 Holes Specified ^{(2) (3)}
14"	20"	Mid (2nd hole from bottom)	6" (E)	1st and 2nd hole from bottom
14"	22"	Low (3rd hole from bottom)	6" (E)	1st and 2nd hole from bottom
15"	20"	High- Standard (1st hole from bottom)	6" (D) or 8" (E)	1st and 2nd hole from bottom
15"	22"	Mid- Standard (2nd hole from bottom)	6" (D) or 8" (E)	1st and 2nd hole from bottom
15"	24"	Low- Standard (3rd hole from bottom)	6" (D) or 8" (E)	1st and 2nd hole from bottom
15"	20"	Mid (2nd hole from bottom)	6" (E)	2nd and 3rd hole from bottom
15"	22"	Low (3rd hole from bottom)	6" (E)	2nd and 3rd hole from bottom
16"	20"	High- Standard (1st hole from bottom)	6" (D) or 8" (E)	2nd and 3rd hole from bottom
16"	22"	Mid- Standard (2nd hole from bottom)	6" (D) or 8" (E)	2nd and 3rd hole from bottom
16"	24"	Low- Standard (3rd hole from bottom)	6" (D) or 8" (E)	2nd and 3rd hole from bottom
16"	20"	Mid (2nd hole from bottom)	6" (E)	3rd and 4th hole from bottom
16"	22"	Low (3rd hole from bottom)	6" (E)	3rd and 4th hole from bottom
16"	22"	High (1st hole from bottom)	6" (B) or 8" (D)	1st and 2nd hole from bottom
16"	24"	Mid (2nd hole from bottom)	6" (B) or 8" (D)	1st and 2nd hole from bottom
17"	20"	High- Standard (1st hole from bottom)	6" (D) or 8" (E)	3rd and 4th hole from bottom
17"	22"	Mid- Standard (2nd hole from bottom)	6" (D) or 8" (E)	3rd and 4th hole from bottom
17"	24"	Low- Standard (3rd hole from bottom)	6" (D) or 8" (E)	3rd and 4th hole from bottom
17"	20"	Mid (2nd hole from bottom)	6" (E)	4th and 5th hole from bottom
17"	22"	Low (3rd hole from bottom)	6" (E)	4th and 5th hole from bottom
17"	22"	High (1st hole from bottom)	6" (B) or 8" (D)	2nd and 3rd hole from bottom
17"	24"	Mid (2nd hole from bottom)	6" (B) or 8" (D)	2nd and 3rd hole from bottom
17"	24"	High (1st hole from bottom)	8" (B)	1st and 2nd hole from bottom
18"	20"	High- Standard (1st hole from bottom)	6" (D) or 8" (E)	4th and 5th hole from bottom
18"	22"	Mid- Standard (2nd hole from bottom)	6" (D) or 8" (E)	4th and 5th hole from bottom
18"	24"	Low- Standard (3rd hole from bottom)	6" (D) or 8" (E)	4th and 5th hole from bottom
18"	22"	High (1st hole from bottom)	6" (B) or 8" (D)	3rd and 4th hole from bottom
18"	24"	Mid (2nd hole from bottom)	6" (B) or 8" (D)	3rd and 4th hole from bottom
18"	24"	High (1st hole from bottom)	8" (B)	2nd and 3rd hole from bottom
19"	20"	High (1st hole from bottom)	6" (B) or 8" (D)	4th and 5th hole from bottom
19"	22"	High (1st hole from bottom)	6" (B) or 8" (D)	4th and 5th hole from bottom
19"	24"	Mid (2nd hole from bottom)	6" (B) or 8" (D)	4th and 5th hole from bottom
19"	24"	High (1st hole from bottom)	8" (B)	3rd and 4th hole from bottom
20"	24"	High (1st hole from bottom)	8" (B)	4th and 5th hole from bottom

⁽¹⁾ Seat height specified is for the seat pan. Add 1" to the specified seat height if a strapped seat is used.

⁽²⁾ Lower hole is for the cylinder mounting bolt, upper hole is for the seat height post securement bolt.

⁽³⁾ Both bolts **MUST** pass through the seat height post **AND** also the exterior tube that supports the seat height post.

