



Parts Required: Side Arm Support – color specific

Step	Execution	Figures
1	There are three attachment straps with D-rings located on the outer back side of the top right hand side of the pad. These are to be attached to the back canes, while the chair is in an upright position. Position the pad where desired, to fill the gap between the strapped wing frame and the arm frame, and secure straps & D-rings.	
2	There is a velcro strap attachment located on the outer back side on the top left hand corner. Attach this strap to the removable wing frame to secure in position as shown.	

For more information, please contact Broda Customer Care at 1-800-668-0637