




Instruction: Neckrest/Headrest Installation

Approval: Rob


Document Number:	SWI-07-080
Revision No.:	0
Revision Date:	11-Jun-12
Written By:	Ashley

Step	Tools	Qty	Hardware	Qty	Broda Part	Execution	Figures
Step 1						<p>If the chair has removable wings and shoulder bolsters, please remove from chair, for easiest accessibility before attaching the neckrest/headrest.</p>	
Step 2						<p>Position the neckrest/headrest near the top of the back frame.</p>	
Step 3						<p>Wrap each strap around the back canes and secure buckles.</p>	

Instruction: Neckrest/Headrest Installation

Approval: Rob

Document Number:	SWI-07-080
Revision No.:	0
Revision Date:	11-Jun-12
Written By:	Ashley

Step 4						Tighten the straps as desired.	
Step 5						Please Contact Broda Customer Service at 800-668-0637 with questions.	

<u>Hardware List</u>		
Check	QTY	ITEM