
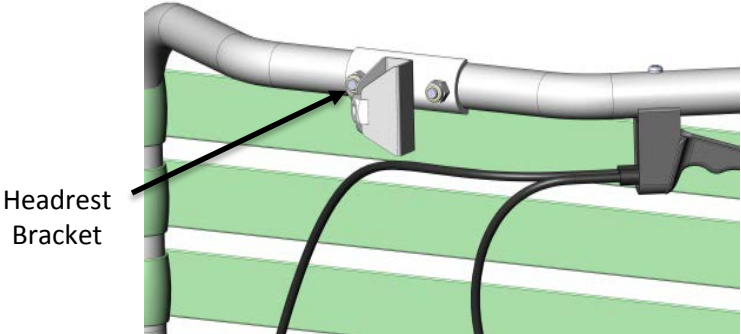
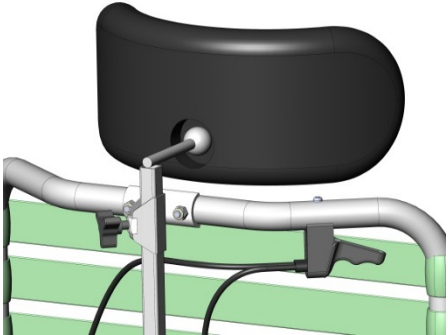


Parts Required: Molded Headrest
Headrest Bracket
3 Point Handle

Tools Used: #2
Robertson
7/16" Wrench

Step	Execution	Figures
1	Position the headrest bracket to line up with the mounting holes on the backrest frame and secure using the 2 x 1/4-20x1-1/2 Stainless Screws & Hex Nuts.	
2	Slide the stem of the headrest through the bracket and position at desired height.	
3	Insert 3 Point Handle through mounting hole on headrest bracket and tighten. Re-adjust height as needed by loosening the 3 Point Handle and tightening once headrest has reached desired height. See operating manual to operate angle adjustments.	

For more information, please contact Broda Customer Care at 1-800-668-0637

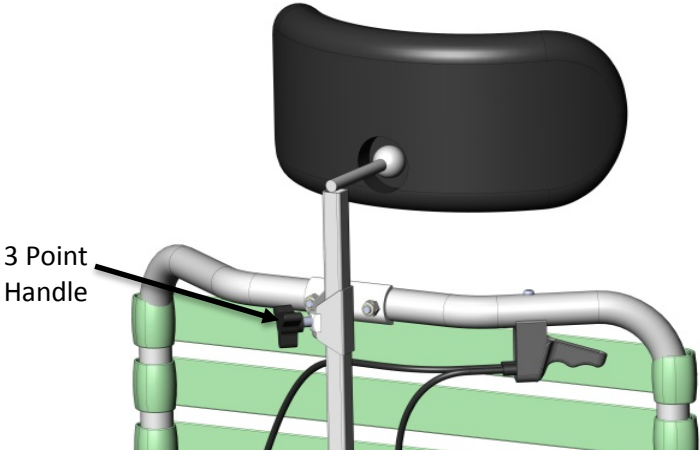