






Instruction: Seat Pad Installation

Approval: Rob Deans

Document Number:	SWI-07-089
Revision No.:	0
Revision Date:	29-Aug-12
Written By:	Ashley

Step	Tools	Qty	Hardware	Qty	Broda Part	Execution	Figures
Step 1						Position your seat pad and cover on top of the seat frame with the "V" Notch of the pad towards the front of the seat.	 <p>"V" Notch</p>
Step 2						Tuck all four straps down the sides of the chair, between the seat frame and arm frame.	 <p>Between the seat frame and arm frame.</p>
Step 3						Step # 2 - Picture Continued	
Step 4						Secure both sets of straps and buckles together underneath seat frame. Tighten as desired. *Note: for 785 models, the straps should be secured underneath all moving parts to avoid interference.	 <p>Secure buckles</p>
Step 5						Step # 4 - Picture Continued	 <p>Both buckles are secured under the seat</p>
						Please contact Broda Customer Service at 866-668-0637 for questions or concerns.	