






**Parts Required:** Padded Seat Belt

Step	Execution	Figures
1	Place the seatbelt on the resident in their seated position in the chair, with the padded portion centered over the resident's midsection.	
2	Connect the strap buckles behind the chair. Do not fasten around the cable or cylinder behind the chair. Tighten the straps as necessary.	
3	When using a seat belt, always ensure the resident is in a supervised area and is correctly positioned as to avoid sliding under the seatbelt	
4	Auto Style buckle also available. Repeat steps 1-3	
<b>For more information, please contact Broda Customer Care at 1-800-668-0637</b>		