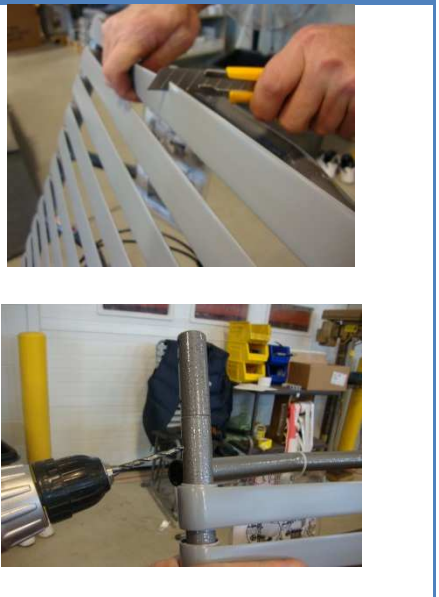

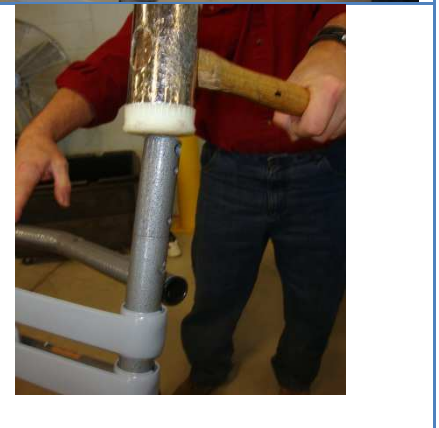
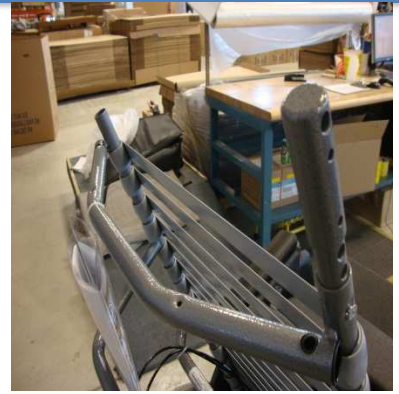


**Parts Required:** Plugs  
Vinyl Straps (width & color specific)

**Tools Used:** 3/16" Drill  
Rubber Mallet  
Screw Driver

Step	Execution	Figures
1	Remove the top strap from the back by cutting the strap and drilling out the rivets that hold it in place with a 3/16" drill.	
2	Remove the black plastic plugs at the top of the back (a screw driver works well to pry out the plugs)	
3	Using a soft tipped hammer (rubber mallet), install your back height extension plugs into the open tubes at the top of back height. Note, pass holes in the plugs must line up with the pass holes in the back tubes. (If they do not line up, use a screw driver to line them up)	



- 4** Install new replacement straps (width and color specific). 2 Straps required for 1.5" extension and 3 straps required for 3" extension. See Vinyl Strap Installation Instructions if required.



- 5** Picture shows finished 3" back height extension



**For more information, please contact Broda Customer Care at 1-800-668-0637**