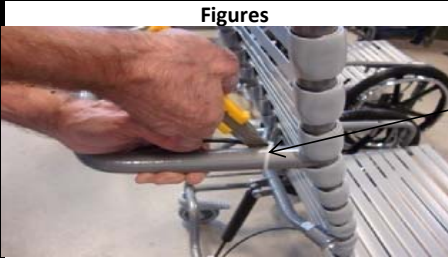




Instruction: Cabhand Installation

Approval: Travis Clark

Document Number:	SWI-07-024
Revision No.:	1
Revision Date:	5-Oct-16
Written By:	Ashley Myer

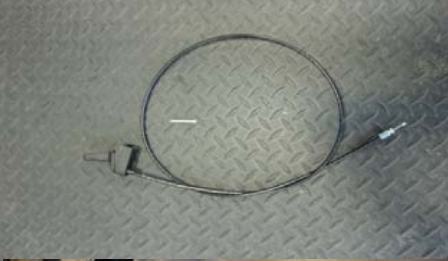
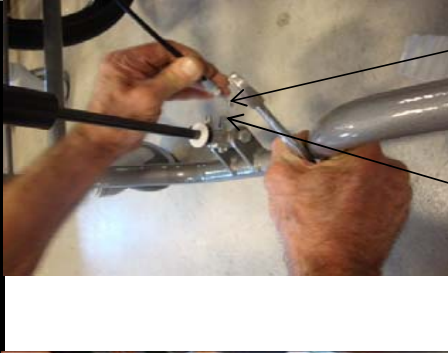

Step	Tools	Qty	Hardware	Qty	Broda Part	Execution	Figures
Step 1						Cut the cable ties that secure the cable to the frame of the chair (note the location of the cable ties and how the cable is secured to the frame of the chair)	
Step 2	#2 Robertson Screw Driver	1				Remove the mounting screw that attaches the cable and handle to the frame of the chair.	
Step 3	17mm Wrench	1				Loosen nut and rotate the cable clip free of the cable. Remove the end of the cable from the lever, connected to the actuator head. Note: The current style cable clip includes a washer, which does not require the nut to be loosened, please ignore this step and move onto step # 4.	

Cable Tie

Mounting Screw

Nut

Cable Clip

Step 4				<p>Note: Discard any damaged/faulty parts removed in the steps above. Keep hardware for replacement Cabhand if necessary. Your replacement cabhand should include all parts shown in the diagram.</p>			
Step 5			Cabhand	<p>Install the end of the cabhand to the lever, the same way it was removed in step 3. Reposition the cable clip, secure over the cable and tighten nut.</p> <p>Note: The current style cable clip includes a washer, which does not require the nut to be loosened, please ignore this step.</p>		End of Cable	Lever
Step 6	#2 Robertson Screw Driver		Cabhand	<p>Attach the new handle end of the Cabhand assembly to the same position it was removed from in Step 2. Tighten the screw with a # 2 "Robertson" screw driver.</p>			
Step 8				<p>Please Contact Broda Customer Service For Questions; 1-800-668-0637</p>			

Hardware List		
Check	QTY	ITEM
	1	#2 Robertson Screw Driver
	1	17 mm Wrench