
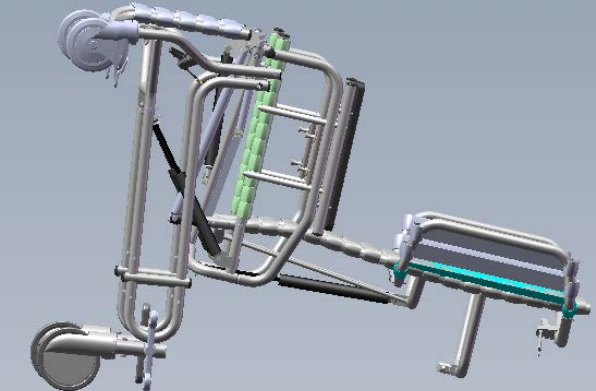
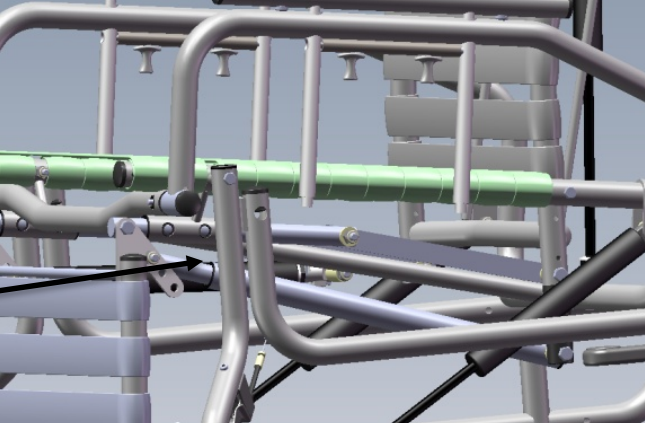
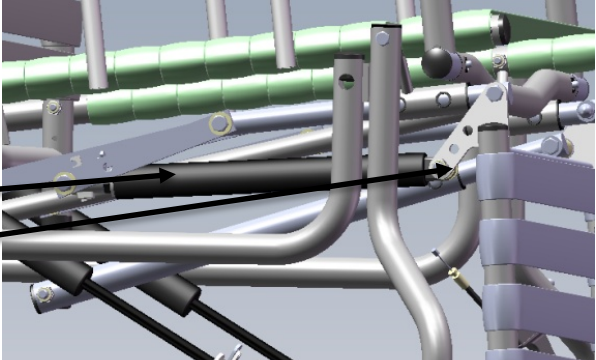


Parts Required: Legrest Frame (width & strap color specific)

Tools Used:
 ½" Wrench

Step	Execution	Figures
1	Use existing hardware, see list at bottom of instruction. Note: The elevation rod is date and size specific, be sure to supply a serial upon ordering a replacement.	
2	Remove the footrest from the legrest. It may be easier to position your chair on the back, with the legrest in the air, as shown in the picture.	
3	**Before removing any hardware, keep in mind the sequence of hardware, for the installation of the replacement legrest.** Standing at the front of the chair, the elevation rod is on the right hand side, positioned under the attachment rod. Using two 1/2" wrenches remove the 5/16-18 hex nut from the outer side of the elevation rod. Remove elevation rod from legrest. *Nut that attaches elevation rod	
4	Standing at the front of the chair, the PAB5 cylinder is positioned on the left hand side of the legrest. Using two 1/2" wrenches, remove the nut from the outer side of the cylinder. Remove the cylinder from the legrest. *PAB 300N Cylinder *Nut that attaches cylinder to legrest	


- 5** There are two attachment rods, one on the left and one on the right side of the legrest. Using two 1/2" wrenches, remove the 5/16" acorn nuts, then tap out the 5/16 - 1 1/2" bolts, completely removing the legrest from the chair. (Keep in mind the sequence of the bolts and washers)

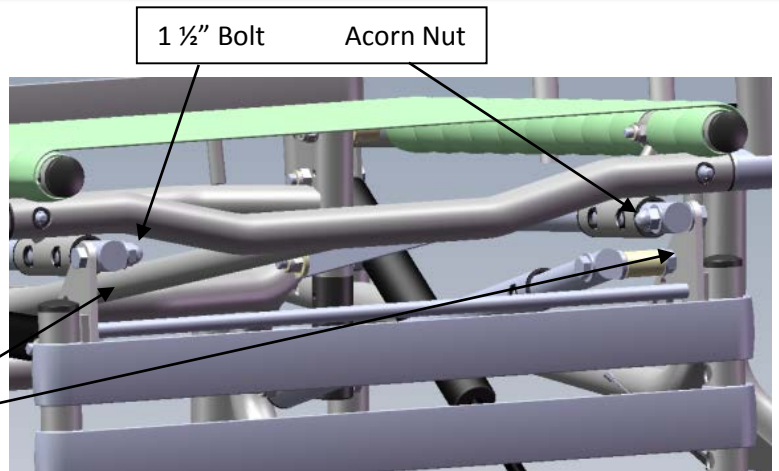

- 6** Slide the attachment rods through the legrest guide tube bushings found underneath both sides of the seat. Install the two attachment rods in the top hole of the flange tab using two 1/2" wrenches, the same way it was removed in step # 5




- 7** Install the replacement legrest by sliding one washer on the bolt, assemble the bolt through the elevation rod, slide on a second washer, spacer, then third washer and slide through the second hole of the flange tab. Use two 1/2" wrenches and using the same sequence of hardware, as removed in step # 3.

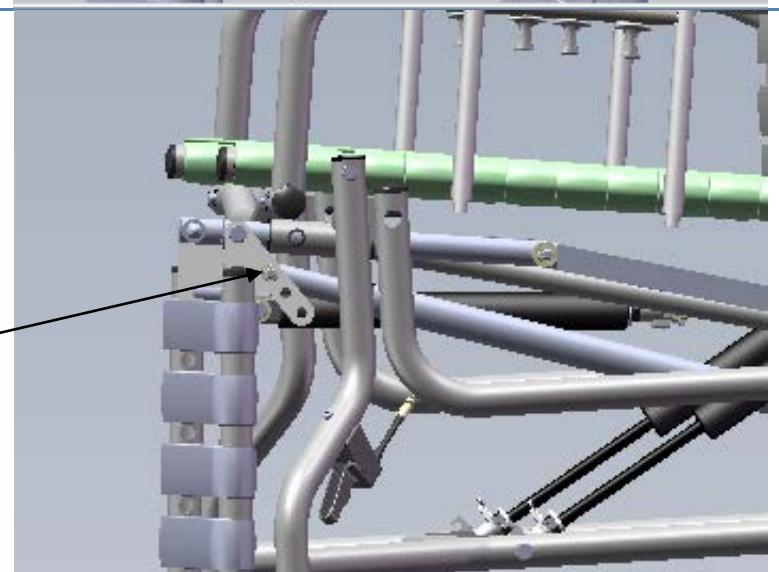

- 8** Using two 1/2" wrenches, install the PAB5 cylinder to the bottom hole in the flange tab, the same way it was removed in step # 4.

Bottom hole in flange tab

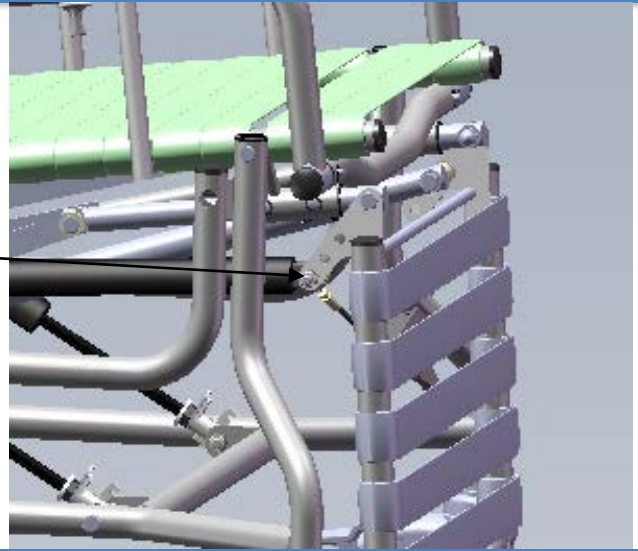

For more information, please contact Broda Customer Care at 1-800-668-0637