
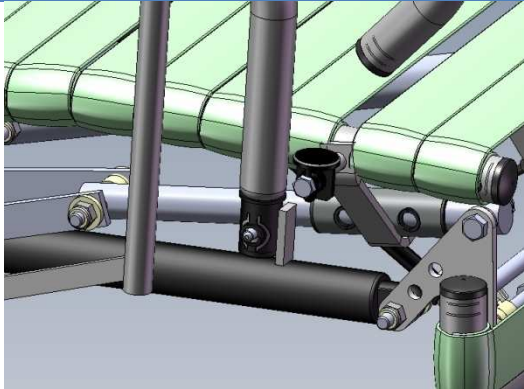
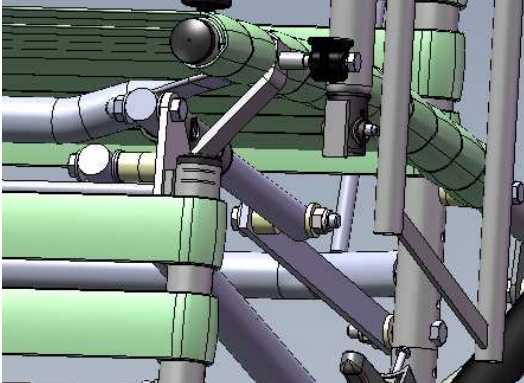





Parts Required: 2 x Attachment Rods (Serial Number & Depth Specific)

Tools Used: ½" Wrench

Step	Execution	Figures
1	Use existing hardware (keeping in mind the sequence), see list at bottom of instruction. Remove footrest from chair.	
2	Standing at the front of the chair, the attachment rod is attached to the J-Lever and extension rod on the left hand side. Use two 1/2" wrenches to unbolt the 5/16" - 18x2 - 1/4" long hex cap screw from the 5/16 - 18 nylon insert locknut, remove the attachment rod from the J-Lever and extension bar.	
3	Standing at the front of the chair, the attachment rod is attached to the extension rod only, on the right hand side. Using two 1/2" wrenches, unbolt the 5/16" - 18x2 - 1/4" long hex cap screw from the 5/16 - 18 nylon insert locknut, removing it from the extension rod. Leave the extension rods and J-lever hanging there, ready for installation of the replacement attachment rods, do not completely remove.	



- 4** The two attachment rods are also attached to the top hole of the legrest flange tabs on the left and right side of the chair. Using two 1/2" wrenches, remove the 5/16" acorn nuts, then tap out the 5/16 - 1 1/2" bolts, completely removing the attachment rods, by sliding them out of the legrest guide tube bushings. (Leave the legrest attached to the elevation rod and cylinder, do not remove)

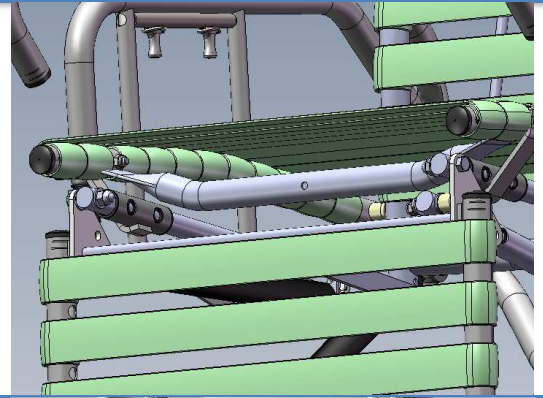

- 5** Attach the replacement attachment rods, the same way they were removed, using the same sequence of hardware.



For more information, please contact Broda Customer Care at 1-800-668-0637