
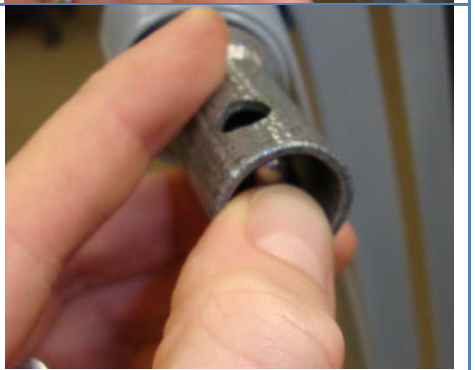



**Parts Required:** Detent Pin

Step	Execution	Figures
<b>1</b>	Squeeze the detent pin depressing it enough to allow you to insert in the tube. place the pointed angular end of the detent pin into the tube, with the push point facing the punch hold provided in the tube.	
<b>2</b>	Insert the detent pin far enough in the tube, until the push point pops out of the hole in the tube.	
<b>3</b>	Once completed, the detent pin should look as shown in the diagram on the right.	

**For more information, please contact Broda Customer Care at 1-800-668-0637**