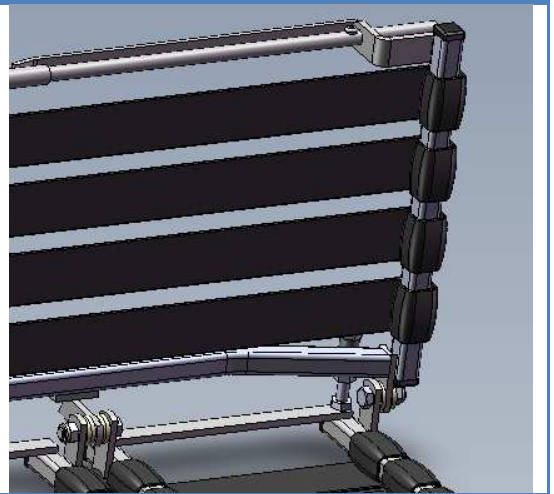

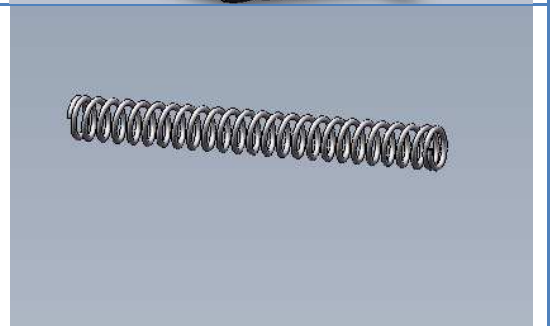


**Parts Required:** F-ROD Foot Rod (width specific)  
F-SPRING Footrest Spring 3.25" Long

**Tools Used:** #2 Robinson  
Phillips Screwdriver

Step	Execution	Figures
<b>1</b>	Loose the #8 x 3/8 Pan quad self-tapping screw attached to the footrest rod using a #2 Robinson or Phillips screw driver to completely remove the defective footrest rod. (Set screw aside for re-use)	
<b>2</b>	Slightly turn the footrest on an angle to slide the footrest spring out of the frame portion, where the footrest rod was inserted and discard.	
<b>3</b>	Slide the replacement footrest rod into the cylinder tube, with the hole facing upwards, the same way it was removed in Step # 1.	

- 4** Slide the replacement footrest rod into the cylinder tube, with the hole facing upwards, the same way it was removed in Step # 1.



- 5** Insert and tighten the #8 x 3/8 Pan quad self tapping screw the same way it was removed in Step # 1.



**For more information, please contact Broda Customer Care at 1-800-668-0637**