


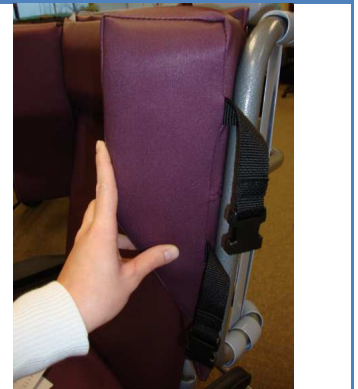

**Parts Required:** SB-SH Shoulder Bolster Short Pad (color specific)

Step	Execution	Figures
1	Position the shoulder bolster in front of the strapped wing frame.	
2	Wrap the top strap with the male buckle around the frame portion of the removable wing and wrap the strap with the female buckle around the inside of the removable wing, over the vinyl straps and secure buckle.	
3	Wrap the lower strap with the male buckle through the frame, around the vinyl straps and wrap the female end between back frame and around vinyl straps. Secure buckle. Tighten as desired.	
<p style="text-align: center;"><b>For more information, please contact Broda Customer Care at 1-800-668-0637</b></p>		