






**Parts Required:** Model MID chair with 1/4" seat pivot bolts (Built before 2016-AUG-01)  
 Replacement base for MID with 5/16" seat pivot holes (width specific)  
 Seat pivot bolts, nuts, and spacers specified on drawing MIDLINE CHAIR ASM, MIDLINE CHAIR ASM OVR, or MIDLINE CHAIR ASM IND LEG

**Tools Used:** 7/16" Wrench (X2)  
 1/2" Wrench (X2)  
 #2 (Red) Robertson Screwdriver  
 5/16" Drill Bit  
 Drill

Step	Execution	Figures
1	MID Base is width specific. See illustration for an example. Contact your sales representative for configuration verification.	
2	Remove original base from chair (keep fasteners for reuse).	
3	Remove armrests and armrest locking spring-plunger components from base.	
4	Remove legrest elevation cable handle from base.	

- 5 Remove seat tilt cylinder bolts from base.

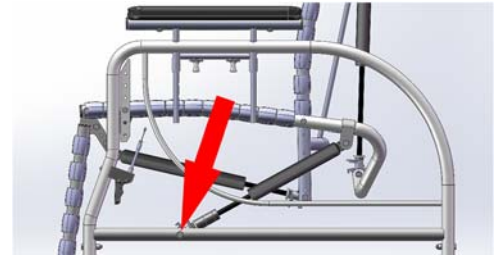

- 6 Remove seat pivot bolts from base seat height tab. (Note which hole is being used) (Discard these fasteners and spacers.)



- 7 Remove all four casters from base. (Note right rear caster is a directional locking tracking caster.)



- 8 Drill out seat pivot holes in original seat to 5/16". Be careful to drill straight through the existing holes without drifting on an angle.



- 9 Reassemble chair with new base and new seat pivot hardware (i.e., bolts, nuts, spacers), as specified in applicable drawing, in reverse order described above.



**For more information, please contact Broda Customer Care at 1-800-668-0637**